


Métier Medical TM

Safe-T-Clip

Safety Hypodermic Needle

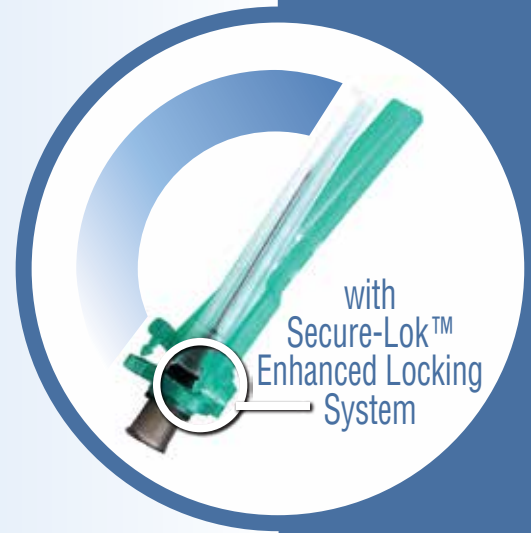

Product Data & Information



- ▲ Métier Medical Limited certifies that the Safe-T-Clip™ safety needle fulfills the requirements of the United States of America Food & Drug Administration (FDA) Quality Systems Regulations (21 CFR 820), the Medical Device Directives (93/42/EEC), and ISO 13485:2003.
- ▲ Safe-T-Clip™ safety needles are available in the following needle gauges: 18, 19, 20, 21, 22, 23, 25, 26, 27, 29, and 30 Gauge; with a range of needle lengths from ½" to 1 ½".
- ▲ Safe-T-Clip™ safety needles contain no rubber latex, they are non-toxic, pyrogen free, and single use.
- ▲ Safe-T-Clip™ safety needles are individually wrapped in a peel apart, ISO needle color-coded package, which is marked with a Lot number and expiration date.

How to use the Safe-T-Clip™ Safety Needle for Luer Lock or Luer Slip Syringes:

01. Attachment

- ▲ Step 01: Securely attach needle onto a Luer Lock or Luer Slip syringe.
- ▲ Step 02: Pull the safety cover back away from the needle and toward the syringe barrel.
- ▲ Step 03: Grasp the syringe with one hand and with the other hand pull the clear needle shield straight off.

02. Aspiration

- ▲ Draw up medication as per your established protocol.
- ▲ NOTE: Changing of needles is possible, use established institutional protocol; do not activate safety cover until after the injection has been given, prior to removing the needle from the syringe.

03. Injection

- ▲ Administer medication in accordance with established protocol.

04. Activation

- After injection, immediately activate safety cover as follows:
- ▲ Without changing hand position, move the cover to a halfway position with a finger motion then activate by depressing on a flat surface. An audible click indicates the safety mechanism has been activated.
Or
 - ▲ Push the safety needle cover forward toward the needle until you hear and/or feel it lock. Visually confirm that the needle tip is covered.
 - ▲ Discard after single use in an approved sharps container.


Métier Medical TM
Safe-Clip
 Safety Hypodermic Needle

Safety Needle Packaging Dimensions / Cases

50 Units per box	19 x 10 x 9 cm
2500 Units per Shipping Case	52 x 40 x 48 cm
20ft Container = 800,000 pcs	320 Shipping Cases
MOQ = 100,000 pcs	40 Cases

Shelf Carton

(Information and format may vary based on lot numbers and manufacturing site)

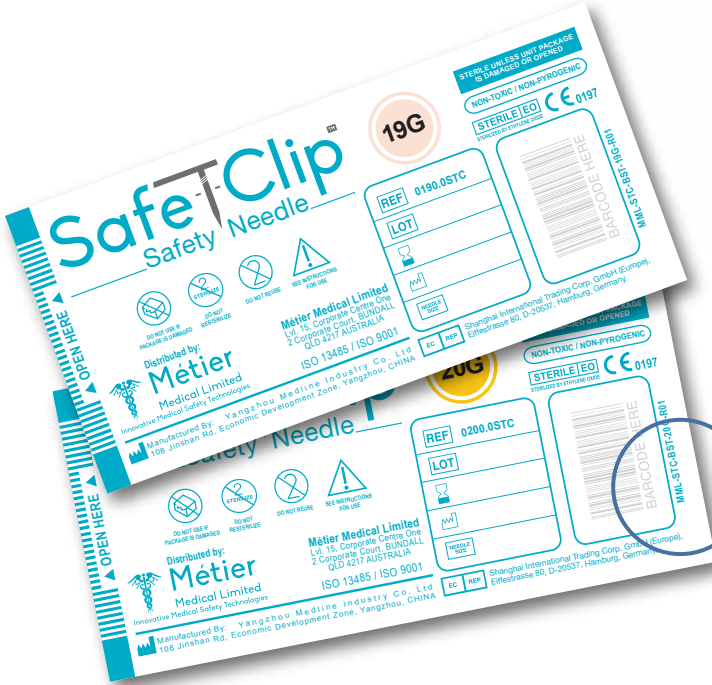

Individually Packaged

(Information and format may vary based on lot numbers and manufacturing site)



Métier
 Medical Limited

Innovative Medical Safety Technologies
 WWW.METIERMEDICAL.COM

Lvl. 15, Corporate Centre One
 2 Corporate Court, BUNDALL
 QLD 4217 AUSTRALIA

Tel: +61 7 5668 5436
 Tel: +61 2 4909 8038
 Email: info@metiermedical.com