

# Instructions for Use

## Step 01

Attach the Safe-T-Clip™ needle to any Luer Lock or Luer Slip syringe.



## Step 02

Pull the safety cover back so that it sits parallel to the syringe barrel allowing for a clear view of the injection site. Remove the needle shield.



## Step 03

Draw up medication in accordance with established protocol.



## Step 04

Administer medication according to established protocol.



## Step 05

Immediately following injection, the safety cover can be locked over the needle by depressing on a flat surface (an audible click indicates the safety mechanism has been activated). Discard in an approved sharps waste container.

